

Aldehyde Free

# Mi-Guard<sup>®</sup>

High Level Disinfectant for  
Critical & Non-Critical Surfaces



- 5th Generation QAC
- Sporicidal • Myco-bactericidal
- Stain removing • Surface cleaning
- Wet wiping
- Corrosion free
- Perfumed for fresh aroma
- Works in hostile conditions
- Bio-degradable

**ACTIVE INGREDIENTS**

Alkyl Dimethyl Benzyl Ammonium Chloride (QAC) 13.8% w/v  
Didecyl Dimethyl Ammonium Chloride (QAC) 9.2% w/v



**Excellent Disinfectant Property with Broad Spectrum Coverage**



# Mi-Guard®

High Level Disinfectant for Critical & Non-Critical Surfaces

Dilution for Use : 3.9 ml in 1 Liter cold water



## AREAS OF APPLICATION :

Mi-Guard is a high level disinfectant with sporicidal properties. It is ideal for high level disinfection of critical & non critical surfaces in areas like OT, ICU, Nurseries, Transplant Units, Burn Units, Oncology units, HDUs, Eye & Dialysis Centers etc. It is also suitable for disinfecting highly contaminated areas like mortuaries, dirty utility room and objects like bed pans, biomedical waste etc.

Mi-Guard is an excellent choice for fabric and lines disinfection as well.



## DIRECTIONS FOR USE :

Pre clean heavily solid areas

**Surface Mopping :** Mix 3.9ml Mi-Guard solution in 1 liter cold water (max. room temperature). Use this solution for mopping. Use this solution for spraying with power sprayer if required.



Apply Mi-Guard solution to hard, non porous environmental surfaces. All surfaces must remain wet for 10 minutes for disinfectant.

**Fabric & Linen Disinfection:** Mix 3.9ml Mi-Guard solution in 1 liter cold water (max. room temperature). Use this solution for manual washing. For automated wash, add to fabric conditioner dispenser.

Use freshly prepared solution for best results.

## PRECAUTIONS :

- For external use only.
- Avoid direct contact with eyes.
- Flush eyes with fresh water in case of contact.
- In case of emergency seek medical help.
- Please ensure your employees must read and understand MSDS before using Mi-Guard.
- Contact with Aldehyde based products must be avoided. If disinfection was previously carried out using aldehyde based products intermediate cleaning is recommended before changing to Mi-Guard.

Pack Size  
1 Liter



**FAITH MICROSOLUTIONS**

ISO 9001 : 2015 & ISO 13485 : 2016 Company

Head office: 27, Ashoka Chambers, 5-B Rajendra Park,  
New Delhi-110060, INDIA

Phone : 011-47479797 (20 Lines)

Customer Care : info@faithmicrosolutions.com

Url : www.faithmicrosolutions.com

Authorised Representative :

Connect with us at :

