

Mi-Rapid Spray®

Aldehyde Free, Alcohol Based, Fast Acting, Rapid Disinfectant



Active ingredients :

Didecyl dimethyl ammonium chloride	0.082% w/v
Octyl decyl dimethyl ammonium chloride	0.13% w/v
Diocetyl dimethyl ammonium chloride	0.057% w/v
Benzalkonium chloride I.P.	0.3% w/v
Isopropyl Alcohol I.P.	45% v/v
N Propanol	35% v/v
Sodium benzoate	0.01% w/v
Delonised water	qs

Direction for use

Spray from 6-8 inches from surface, until surface are thoroughly wet. Wipe dry with a clean cloth, sponge or mop or allow to air dry.

- ◆ Rapid Action
- ◆ Quick Evaporation
- ◆ Deodorises
- ◆ Residual Action
- ◆ Bactericidal
- ◆ Fungicidal
- ◆ Virucidal
- ◆ Effective against all types of bacteria, viruses & fungi like H1N1 Influenza A, Staphylococcus, Pseudomonas
- ◆ Biodegradable



One step disinfectant - Just spray & wipe for excellent results

Mi-Rapid Spray®

Aldehyde Free, Alcohol Based, Fast Acting, Rapid Disinfectant



Areas of application

Aldehyde - free, alcohol based Mi-Rapid Spray is a rapid disinfectant with prolonged residual action. Mi-Rapid Spray is suitable for the rapid disinfection wipe procedure. It is used in situations where rapid effect is required along with fast drying. This makes it ideal for disinfection of medical equipment like ventilators, incubators, monitors, OT lights, OT table, trolleys, beds, cabinets, nursing station, radiological equipment, laboratory equipment etc.

Mi-Rapid Spray is effectively used in

NICU : Pulse Oxymeter, Photo therapy unit, Incubators, Patient warmer, Infusion pump, Syringe pump, Ventilator, Multi para monitor, Bed, Chair, Trolley, Cabinet, Cables, Adaptors, Connectors etc.

ICU : Pulse oxymeter, infusion pump, syringe pump, ventilator, multi para monitor, bed, chair, trolley, cabinet, cables, adaptors, connectors etc.

OT : OT table, OT light, Electro surgical unit, Trolleys, suction unit, OT pendants etc.

Dental Units : Dental chair, Spittoon, Hand pieces, Kidneys trays, Waste bin etc.

Ophthalmic Units: Ophthalmic chair, Microscope, Imaging equipment, Tonometer, Refractor etc.

Hemodialysis : Hemodialysis chair, Hemodialysis system, Blood pump, Respiration monitor etc.

Cath Lab : Image intensifier, Monitors, Light, Shields, Table etc.

Radiology Center : X-Ray, CT, MRI, Ultrasound, Doppler, DEXA, Patient table, Trolley, etc.

Radio-Therapy Unit : Linear Accelerator, Radio Therapy Machine, Imaging equipment, Patient table, Trolley etc.

Mi-Rapid Spray is a Quat based hospital disinfectant that cleans, disinfects and deodorises in ONESTEP.

Broad spectrum : Highly effective bacteria, fungi, mildew & virus.

Rapid: Ready-to-use spray, disinfects in a short time of 30 seconds to 5 minutes.

Complete disinfection: Cleans, disinfects & deodorizes hard surface including vinyl, plastic, glazed ceramic, chrome, porcelain and backed enamel surfaces.

Availability : 500ml bottle

Application	Method
Disinfecting pre-cleaned surface	Spraying
Semi and critical instrument sterilization	Spray on the instrument let it dry on its own, then wipe with sterile cloth
Decontamination of surgical refuge	Spray to cover all the refuge


Precaution :

No switching action if electronic devices are in use while disinfecting.

Hot surfaces must be cooled sufficiently before using MI-Rapid Spray.

Do not breathe spray. Avoid contact with Eyes.

Mi-Rapid Spray is a one step hospital use rapid disinfectant cleaner that is effective against a broad spectrum of bacteria, virus and fungi. It is used on nonporous, hard surfaces to quickly remove dirt, grime, mold, mildew and other common contaminants found in hospitals, nursing homes, clinics medical offices, laboratories, veterinary clinics, food processing plants and pharmaceutical industries.

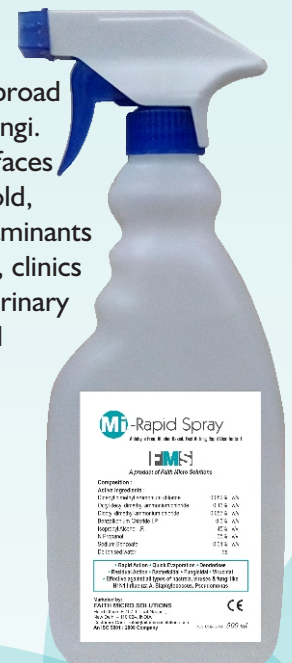

Authorised Representative :



FAITH MICROSOLUTIONS

ISO 9001 : 2015 & ISO 13485 : 2016 Company
 Head office: 27, Ashoka Chambers, 5-B Rajendra Park,
 New Delhi-110060, INDIA
 Phone : 011-47479797 (20 Lines)
 Customer Care : info@faithmicrosolutions.com
 Url : www.faimicrosolutions.com

Connect with us at :

