

Mi-Rust Remover™

Surgical Instrument Rust Remover



- Removes Rust, Stains.
- Restores instruments original finish.
- Restores articulation to box- locks and joints.
- Prevents destructive pitting on ridges of micro surgical and ophthalmic instruments.
- Enhances the life and efficiency of stainless steel.
- Eco- friendly, stable & mild acidic.

Active ingredients :

| | |
|------------------|---------|
| Phosphoric acid | 38% w/v |
| Citric acid | 15% w/v |
| Malic acid | 2% w/v |
| Oxalic acid | 1% w/v |
| DM water to make | q.s. |



FMS
You Say. We Disinfect.

Mi- Rust Remover is a quick, safe and efficient revitalizing instrument Solution

Mi-Rust Remover™

Surgical Instrument Rust Remover

Mi-Rust Remover is highly effective cleaner to remove tarnish loose rust and rusted stain.

It also removes stubborn stains, rust stains & organic residues from Instruments. It is an ideal cleaning agent for surgical instruments made from stainless steel.



Areas of Application:

- CSSD
- OT (where CSSD) is not a separate department,.
- Dental units
- Surgical instruments
- All stainless steel equipments

Direction for use:

- Mix 10 to 15 ml Mi-Rust Remover in 1 liter of water and immerse for 20 minutes. Wash-off with soft water. Dry thoroughly.



Precautions:

- For external use only.
- Avoid direct contact with eyes and skin, rinse with plenty of water in case of contact. Harmful, if inhaled or swallowed. Wear personal protective gear before handling.
- Keep out of reach of children. Seek medical help in case of emergency.

Pack Size : 500 ml.

Authorised Representative :

FMS

You Say. We Disinfect.

FAITH MICROSOLUTIONS

ISO 9001 : 2015 & ISO 13485 : 2016 Company

Head office: 27, Ashoka Chambers, 5-B Rajendra Park,
New Delhi-110060, INDIA

Phone : 011-47479797 (20 Lines)

Customer Care : info@faithmicrosolutions.com

Url : www.faimicrosolutions.com

Connect with us at :

