

Bio - degradable

Mi-Zyme®

Multi Enzymatic Cleaner for Surgical, Medical and Dental Instruments including flexible and rigid Endoscopes



- Highly concentrated stable formulation.
- Breaks down and cleans organic debris.
- Breaks down and cleans inorganic debris.
- Removes Bio-film • Removes stains.
- Low foaming, Neutral pH.
- Excellent material compatibility.
- Cost effective.

Active Ingredients

Mixture of Protease, Amylase, Lipase & Cellulase : 39%



FMS
You Say. We Disinfect.

Bio - degradable

Mi-Zyme®

Multi Enzymatic Cleaner

Highly Concentrated Stable Formulation

Mi-Zyme Multi Enzymatic Cleaner uses a powerful combination of 4 enzymes, i.e. Protease, Amylase, Lipase & Cellulase. These enzymes target different type of debris like proteinaceous matter, tissue, blood, carbohydrates, polysaccharides, saliva, mucous, lipids, fatty substances, creams, ointments, starchy matter, cell wall, fecal matter, medicinal powder etc.

The highly concentrated formulation is the best in class with 39% active ingredients. It provides faster and more thorough penetration into hard to reach at places, making it ideal for cleaning surgical instruments & scopes.

Breaks Down and Cleans both Organic & Inorganic Debris


The 4 enzymes in Mi-Zyme are able to attack and break down organic debris into small pieces and convert the insoluble debris into water soluble debris.

The non-ionic surfactant in Mi-Zyme ensures uniform wetting of surfaces while dissolving the other debris and keeping them in suspension.

Removes Bio-film and Stains

The powerful formulation of Mi-Zyme attacks stains and breaks down bio-film to provide long lasting precise cleaning.

Dilution for Use : 8ml in 1 Liter water

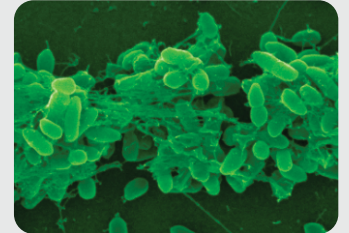

AREAS OF APPLICATION:

Mi-Zyme is highly effective, low foaming, non toxic, biodegradable enzyme formulation. It is ideally suited for pre-cleaning areas contaminated with mold, mildew organic and inorganic contaminants.

Mi-Zyme is used for pre-cleaning of medical devices, surgical and dental instruments. It is also used for linen, clothes, fabric, upholstery and carpet cleaning.

Mi-Zyme removes blood & protein stains. It also removes odour and deodorize while cleaning.

Mi-Zyme is extremely effective in mold and mildew infested areas.

Mi-Zyme can be used for manual, automated & ultrasonic cleaning.

PRECAUTIONS :

- For external use only.
- Avoid direct contact with eyes.
- Flush eyes with fresh water in case of contact.
- Keep out of reach of children.
- Do not add bleach or other chemicals.

DIRECTIONS FOR USE :

Mix 8ml Mi-Zyme solution in 1 liter luke warm water (1oz. Per Gallon). Soak the material to be cleaned minimum for 20 minutes.

Prepare fresh solution for each application.

Thoroughly rinse the equipment, aspirating water through all channels to remove traces of Mi-Zyme.



You Say. We Disinfect.

FAITH MICROSOLUTIONS

ISO 9001 : 2015 & ISO 13485 : 2016 Company

Head office: 27, Ashoka Chambers, 5-B Rajendra Park, New Delhi-110060, INDIA

Phone : 011-47479797 (20 Lines)

Customer Care : info@faithmicrosolutions.com

Url : www.faimicrosolutions.com

Pack Size
1 Ltr.

Authorised Representative :

Connect with us at :

